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MINDRAY BS200e

Test	CA	R1	300
Reac. Type	END POINT	R2	
Pri. Wave	660	Sample Volume	3
Sec. Wave	700	Linearity Range	1 20
Direction	INCREASE	Factor	
Reac. Time	0 12	Slope	1 Intercept 0
Incuba. Time			
Unit	MG/DL		
Precision	0.1		

Calibration	Rule	TWO POINT
	Replicate	2

Test	CHOL	R1	300
Reac. Type	END POINT	R2	
Pri. Wave	505	Sample Volume	3
Sec. Wave	660	Linearity Range	5 500
Direction	INCREASE	Factor	
Reac. Time	0 36	Slope	1 Intercept 0
Incuba. Time			
Unit	MG/DL		
Precision	0		

Calibration	Rule	TWO POINT
	Replicate	2

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MINDRAY BS200e

Test	C3	R1	240			
Reac. Type	END POINT	R2	60			
Pri. Wave	340	Sample Volume	3			
Sec. Wave	700	Linearity Range	10	350		
Direction	INCREASE	Factor				
Reac. Time	-1	18	Slope	1	Intercept	0
Incuba. Time	3					
Unit	MG/DL					
Precision	0					
Calibration		Rule	LOGIT LOG 4P			
		Replicate	2			

Test	C4	R1	240			
Reac. Type	END POINT	R2	60			
Pri. Wave	340	Sample Volume	6			
Sec. Wave	700	Linearity Range	5	90		
Direction	INCREASE	Factor				
Reac. Time	-1	18	Slope	1	Intercept	0
Incuba. Time	3					
Unit	MG/DL					
Precision	0.1					
Calibration		Rule	LOGIT LOG 4P			
		Replicate	2			

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MINDRAY BS200e

Test	COPPER	R1	300
Reac. Type	END POINT	R2	
Pri. Wave	570	Sample Volume	15
Sec. Wave	700	Linearity Range	16 500
Direction	INCREASE	Factor	
Reac. Time	0 12	Slope	1 Intercept 0
Incuba. Time			
Unit	UG/DL		
Precision	0		

Calibration	Rule	TWO POINT
	Replicate	2

Test	CREAT	R1	150
Reac. Type	FIX TIME	R2	150
Pri. Wave	505	Sample Volume	30
Sec. Wave	660	Linearity Range	0.2 15
Direction	INCREASE	Factor	
Reac. Time	4 9	Slope	1 Intercept 0
Incuba. Time			
Unit	MG/DL		
Precision	0.01		

Calibration	Rule	TWO POINT
	Replicate	2

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MINDRAY BS200e

Test	CRP	R1	240
Reac. Type	END POINT	R2	60
Pri. Wave	605	Sample Volume	3
Sec. Wave		Linearity Range	2 100
Direction	INCREASE	Factor	
Reac. Time	-1 12	Slope	1 Intercept 0
Incuba. Time			
Unit	MG/L		
Precision	0.1		

Calibration	Rule	Spline (5 point)
	Replicate	2

Test	GGT	R1	200
Reac. Type	KINETIC	R2	50
Pri. Wave	412	Sample Volume	25
Sec. Wave	660	Linearity Range	5 300
Direction	INCREASE	Factor	
Reac. Time	8 18	Slope	1 Intercept 0
Incuba. Time	3		
Unit	U/L		
Precision	0		

Calibration	Rule	TWO POINT
	Replicate	2

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MINDRAY BS200e

Test	GLUCOSE	R1	300
Reac. Type	END POINT	R2	
Pri. Wave	505	Sample Volume	3
Sec. Wave	660	Linearity Range	5 400
Direction	INCREASE	Factor	
Reac. Time	0 36	Slope	1 Intercept 0
Incuba. Time			
Unit	MG/DL		
Precision	0		

Calibration	Rule	TWO POINT
	Replicate	2

Test	HDL-C	R1	240
Reac. Type	END POINT	R2	80
Pri. Wave	605	Sample Volume	3
Sec. Wave	700	Linearity Range	5 100
Direction	INCREASE	Factor	
Reac. Time	-1 18	Slope	1 Intercept 0
Incuba. Time	3		
Unit	MG/DL		
Precision	0		

Calibration	Rule	TWO POINT
	Replicate	2

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MINDRAY BS200e

Test	IgM	R1	240
Reac. Type	END POINT	R2	60
Pri. Wave	340	Sample Volume	3
Sec. Wave	700	Linearity Range	15 300
Direction	INCREASE	Factor	
Reac. Time	-1 18	Slope	1 Intercept 0
Incuba. Time	3		
Unit	MG/DL		
Precision	0		

Calibration	Rule	LOGIT LOG 4P
	Replicate	2

Test	IRON	R1	200
Reac. Type	END POINT	R2	50
Pri. Wave	570	Sample Volume	25
Sec. Wave		Linearity Range	5 400
Direction	INCREASE	Factor	
Reac. Time	-1 18	Slope	1 Intercept 0
Incuba. Time	3		
Unit	UG/DL		
Precision	0		

Calibration	Rule	TWO POINT
	Replicate	2

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MINDRAY BS200e

Test	LACTATE	R1	300
Reac. Type	END POINT	R2	
Pri. Wave	505	Sample Volume	3
Sec. Wave	660	Linearity Range	2 150
Direction	INCREASE	Factor	
Reac. Time	-1 12	Slope	1 Intercept 0
Incuba. Time			
Unit	MG/DL		
Precision	0		

Calibration	Rule	TWO POINT
	Replicate	2

Test	LDH	R1	200
Reac. Type	KINETIC	R2	50
Pri. Wave	340	Sample Volume	5
Sec. Wave	412	Linearity Range	10 1600
Direction	DECREASE	Factor	
Reac. Time	3 13	Slope	1 Intercept 0
Incuba. Time	3		
Unit	U/L		
Precision	0		

Calibration	Rule	TWO POINT
	Replicate	2

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MINDRAY BS200e

Test	LDL-C	R1	240
Reac. Type	END POINT	R2	80
Pri. Wave	605	Sample Volume	3
Sec. Wave	700	Linearity Range	0 1000
Direction	INCREASE	Factor	
Reac. Time	-1 18	Slope	1 Intercept 0
Incuba. Time	3		
Unit	MG/DL		
Precision	0		

Calibration	Rule	TWO POINT
	Replicate	2

Test	LIPASE	R1	250
Reac. Type	KINETIC	R2	50
Pri. Wave	570	Sample Volume	3
Sec. Wave		Linearity Range	5 250
Direction	INCREASE	Factor	
Reac. Time	4 14	Slope	1 Intercept 0
Incuba. Time			
Unit	U/L		
Precision	0		

Calibration	Rule	TWO POINT
	Replicate	2

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MINDRAY BS200e

Test	MG	R1	300
Reac. Type	END POINT	R2	
Pri. Wave	546	Sample Volume	3
Sec. Wave	700	Linearity Range	0.5 5
Direction	INCREASE	Factor	
Reac. Time	0 12	Slope	1 Intercept 0
Incuba. Time			
Unit	MG/DL		
Precision	0.01		

Calibration	Rule	TWO POINT
	Replicate	2

Test	MIC-ALB	R1	200
Reac. Type	END POINT	R2	50
Pri. Wave	340	Sample Volume	5
Sec. Wave	700	Linearity Range	2 290
Direction	INCREASE	Factor	
Reac. Time	-1 18	Slope	1 Intercept 0
Incuba. Time	3		
Unit	MG/L		
Precision	0		

Calibration	Rule	LOGIT LOG 4P
	Replicate	2

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MINDRAY BS200e

Test	PHOS	R1	300
Reac. Type	END POINT	R2	
Pri. Wave	340	Sample Volume	3
Sec. Wave	412	Linearity Range	1.25 25
Direction	INCREASE	Factor	
Reac. Time	0 12	Slope	1 Intercept 0
Incuba. Time			
Unit	MG/DL		
Precision	0.1		

Calibration	Rule	TWO POINT
	Replicate	2

Test	RF	R1	200
Reac. Type	END POINT	R2	50
Pri. Wave	605	Sample Volume	4
Sec. Wave		Linearity Range	3.5 160
Direction	INCREASE	Factor	
Reac. Time	-1 18	Slope	1 Intercept 0
Incuba. Time			
Unit	IU/ML		
Precision	0		

Calibration	Rule	SPLINE (6 P)
	Replicate	2

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Calibration	Rule	TWO POINT
	Replicate	2

Test	TOTAL-PRO	R1	320
Reac. Type	END POINT	R2	
Pri. Wave	546	Sample Volume	8
Sec. Wave	700	Linearity Range	1 14
Direction	INCREASE	Factor	
Reac. Time	0 12	Slope	1 Intercept 0
Incuba. Time			
Unit	G/DL		
Precision	0.1		

Calibration	Rule	TWO POINT
	Replicate	2

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MINDRAY BS200e

Test	TG	R1	300
Reac. Type	END POINT	R2	
Pri. Wave	505	Sample Volume	3
Sec. Wave	660	Linearity Range	5 700
Direction	INCREASE	Factor	
Reac. Time	0 36	Slope	1 Intercept 0
Incuba. Time			
Unit	MG/DL		
Precision	0		

Calibration	Rule	TWO POINT
	Replicate	2

Test	UREA	R1	240
Reac. Type	FIX TIME	R2	60
Pri. Wave	340	Sample Volume	3
Sec. Wave	412	Linearity Range	2 200
Direction	DECREASE	Factor	
Reac. Time	4 9	Slope	1 Intercept 0
Incuba. Time			
Unit	MG/DL		
Precision	0		

Calibration	Rule	TWO POINT
	Replicate	2

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Calibration	Rule	TWO POINT
	Replicate	2

Test	U.PRO	R1	250
Reac. Type	END POINT	R2	125
Pri. Wave	505	Sample Volume	20
Sec. Wave	700	Linearity Range	6.24 2009
Direction	INCREASE	Factor	
Reac. Time	-1 18	Slope	1 Intercept 0
Incuba. Time	3		
Unit	MG/DL		
Precision	0		

Calibration	Rule	SLPINE (6P)
	Replicate	2

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MINDRAY BS200e

Test	ZINC	R1	300
Reac. Type	END POINT	R2	
Pri. Wave	570	Sample Volume	15
Sec. Wave	700	Linearity Range	5.25 500
Direction	INCREASE	Factor	
Reac. Time	0 14	Slope	1 Intercept 0
Incuba. Time			
Unit	UG/DL		
Precision	0		

Calibration	Rule	TWO POINT
	Replicate	2

Test	FERRITINE	R1	180
Reac. Type	FIXED TIME	R2	60
Pri. Wave	570	Sample Volume	15
Sec. Wave	800	Linearity Range	5 700
Direction	INCREASE	Factor	
Reac. Time	3 18	Slope	1 Intercept 0
Incuba. Time	15		
Unit	UG/L		
Precision	1		

Calibration	Rule	LOGIT LOG (6 P)
	Replicate	2

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